


## Blood sampling – cryoglobulins

Tube without additive, red (2)

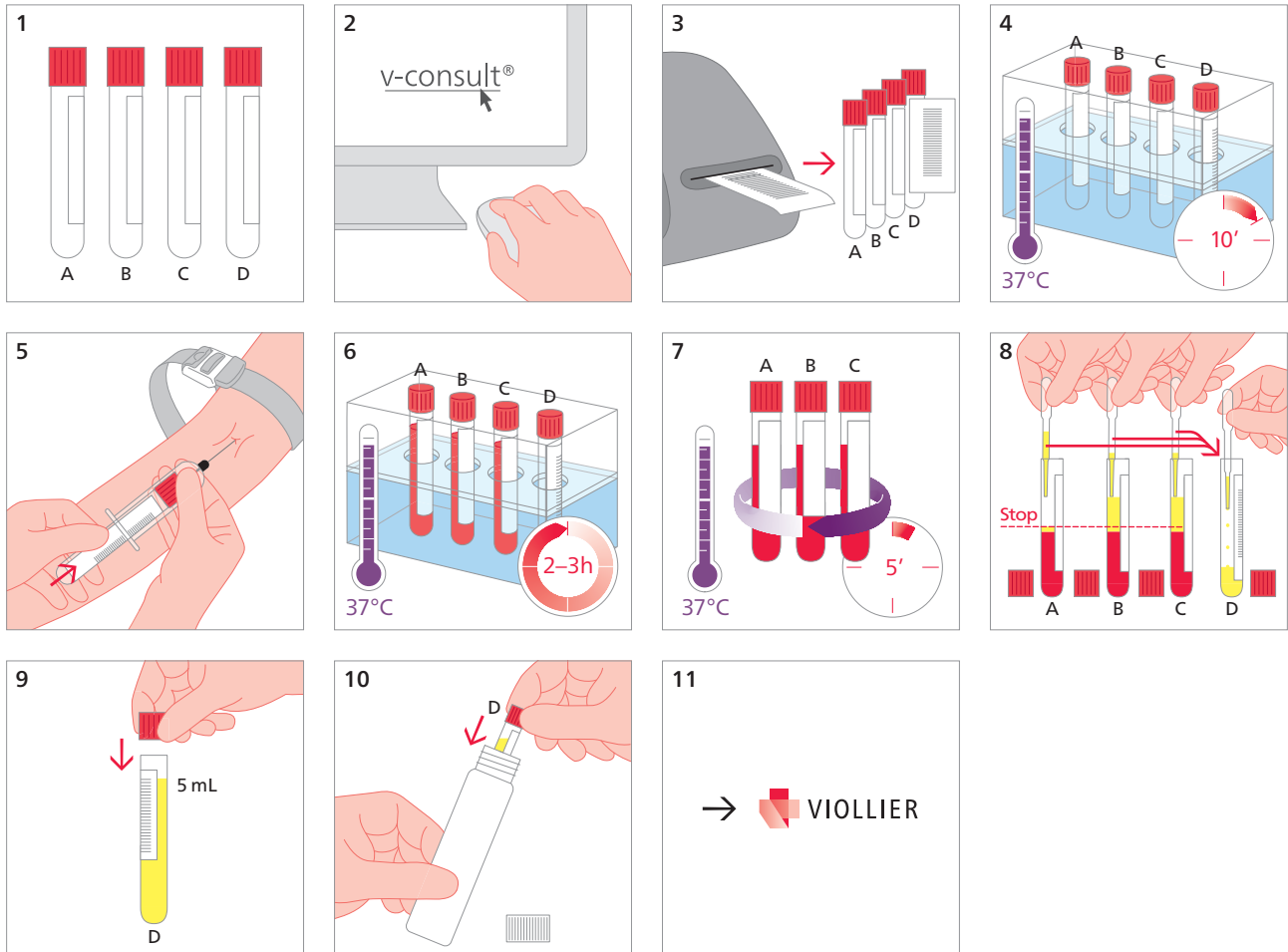

- Preheat the tubes for 10 minutes at 37°C
- Sample in 3 tubes without additive, red
- Incubation 2 – 3h at 37°C (please confirm in v-consult®)
- Centrifugation **must be** at 37°C
- Erythrocytes may alter the result of the analysis
- If this kind of preanalytical procedure may not be guaranteed at the physicians practice, sampling at Viollier may be arranged by phone
- Sampling time: monday – thursday

Shipping material:

Please label with name, first names and date of birth of the patient